


# HITACHI

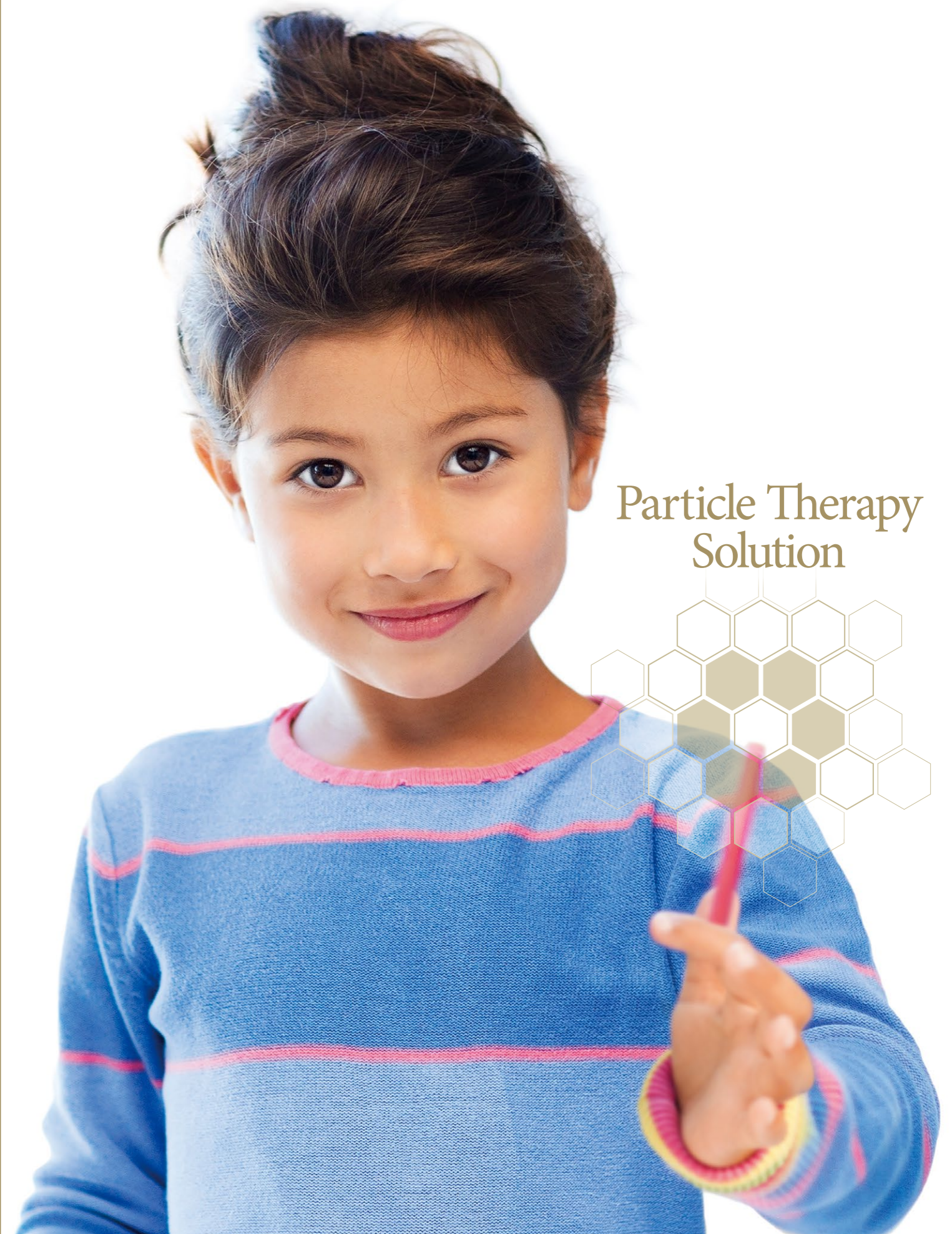

## Particle Therapy Solution

### Hitachi High-Tech Corporation

Toranomon Hills Business Tower, 1-17-1 Toranomon, Minato-ku, Tokyo 105-6409, Japan

For further information about Hitachi High-Tech's particle therapy solution, please refer to the website below.

<https://www.hitachi-hightech.com/global/en/products/healthcare/treatment/pbt/>



Each technology described here needs appropriate approval/clearance/certification by regulatory authorities of each country/region prior to market entry or clinical use.

PBT-E001Q Printed in Japan 2026.03

# Hitachi Particle Therapy Solution


## Particle therapy selected by global leaders

Particle therapy is one of the most advanced forms of cancer treatment. The delivery of high dose to a target with high precision and reduced side effects are the primary qualities that make this therapy so unique. Refining this technology and expanding its accessibility to more cancer patients did not happen overnight. Years of research and development, a broad range of technical and clinical experience, and collaborative work with world-class hospitals and cancer centers have given Hitachi a reputation for providing the medical community with the highest level of quality, exceptionally high clinical availability and cutting-edge innovations in particle therapy.

Discover why leaders in cancer therapy across the globe have selected Hitachi as a long-term partner to help patients fight cancer.

## What is particle therapy?

For over 50 years, the physical advantages of charged particles in cancer therapy have motivated the medical community to advance the clinical application of particle therapy. It can maximize radiation dose to tumor sites while sparing adjacent healthy tissues, and is especially effective in treating many rare cancers, especially pediatric cancers. Particle beams enter the body releasing small amounts of energy until reaching the tumor site, where the particle stops and deposits the vast majority of its energy over a very short distance. As a result, short-term and long-term side effects in uninvolved tissues and organs are reduced or avoided, unlike external beam radiation treatment using X-rays. Given the potential for a more durable cure and better quality-of-life following cancer treatment, it is easy to understand why the number of particle therapy centers and patients seeking particle therapy is steadily increasing.

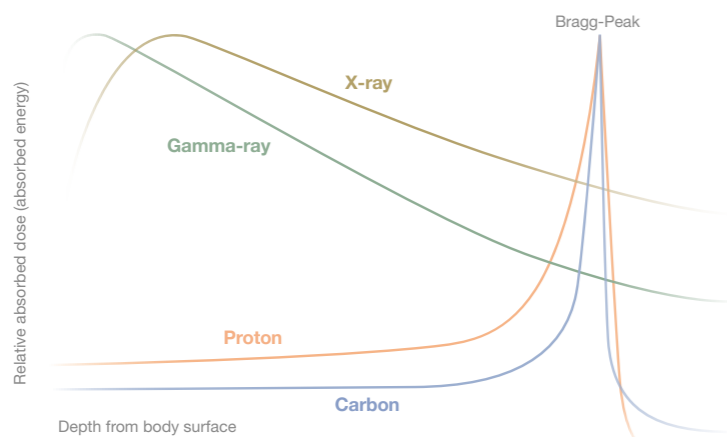

# Ultra High Precision Beam Control Technology

Hitachi's technical advantages are based on its incomparably precise beam control technology – resulting from the continual refinement of electromagnet and synchrotron accelerator design since the 1970's. Leveraging its core strength, Hitachi has collaborated with customers to develop market-leading innovative technology, optimized to meet operational and clinical needs.

## Intensity Modulated Particle Therapy


Hitachi was first to introduce FDA 510(k) cleared and clinically implemented scanning technology to the US market. Scanning beam technology is today's new standard in precision beam delivery. Hitachi's spot scanning system is used to deliver Intensity Modulated Particle Therapy (IMPT) – an advanced form of particle beam treatment. Hitachi particle therapy systems have treated more than 60,000 patients to date, many with Hitachi's industry leading, clinically proven IMPT.

## Real-Time Image Gating for Motion Management

Irradiation of targets that move due to respiration or other factors has posed a challenge, especially with high-precision scanning beams. Hitachi and Hokkaido University collaborated to develop Real-Time Image Gated Particle Therapy (RGPT) – an innovation that enables high dose beam delivery to moving tumors while sparing surrounding healthy tissues and organs.

## Gantry-Mounted CBCT

Precise patient alignment and assessment of anatomical changes require high-resolution imaging of the tumor region at the time of treatment. Gantry-Mounted Cone Beam CT, co-developed with Hokkaido University, provides 3D images of patients in the treatment position. Hitachi's system can also be equipped with a 2-axis CBCT imaging function that uses two pairs of orthogonal X-ray imaging systems at the same time. Compared to 1-axis CBCT imaging, this function has shortened the required imaging time by 35%. It is also useful when it is necessary to shorten the imaging time and when there is a gantry rotation angle limitation due to interference by the patient immobilizer. Alignment software and robotics enable automatic patient positioning - dramatically increasing the precision of irradiation when used with Hitachi's beam control technology. CBCT is a key element in the implementation of adaptive therapy.

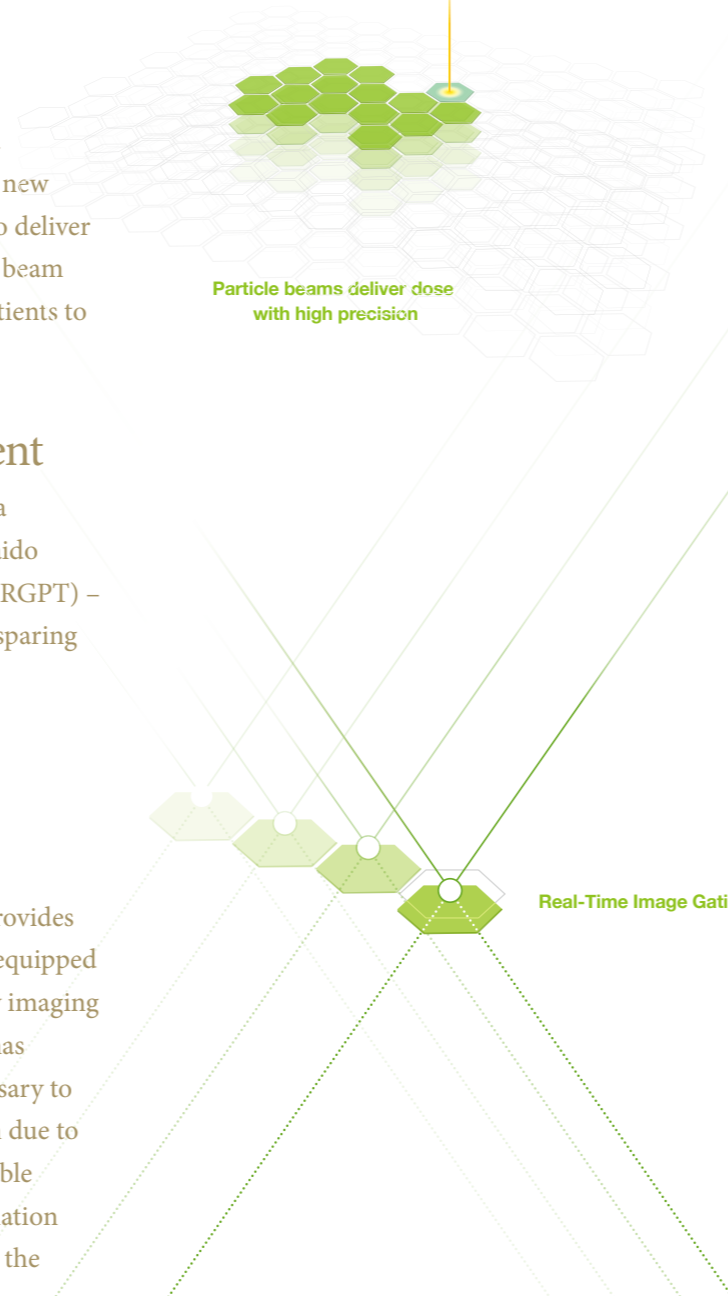

## Adaptive Particle Therapy for Moving Targets

The 2-axis and 4-dimensional CBCT technology, which is expected to be applied to Adaptive Particle Therapy for Moving Targets, combines two technologies co-developed with Hokkaido University—RGPT technology and the technology for simultaneous use of two pairs of orthogonal X-ray imaging systems. Approved by the Japanese Pharmaceuticals and Medical Devices Agency in September 2020, this makes it possible to capture the three-dimensional shape and position of moving targets. With this technology, it is possible to acquire clear three-dimensional images of tumors and surrounding tissues in moving targets, which have been difficult to perceive with two-dimensional X-ray images and conventional CBCT, improving patient positioning accuracy. Hitachi will continue to pursue R&D of this 2-axis and 4-dimensional CBCT technology with the aim of commercialization.

## Power-Efficient and Clean Synchrotron

Hitachi has developed synchrotron accelerators conscious of our customers' needs to operate the system safely and efficiently with peace-of-mind throughout its lifetime.

### Lower power consumption


The synchrotron can supply the exact beam energy required for treatment on demand because it varies the energy electronically. This means it operates more efficiently and consumes less power, lowering operational utility costs.

### Better maintenance access

The synchrotron requires no degrader for energy changes. This drastically reduces neutron generation and enables immediate access to the system for urgent maintenance with almost no waiting time for cool-down.

### Easier decommissioning

Decommission of a particle therapy system may become an issue after many years of use. Since Hitachi's particle therapy system is low in radiation and can keep the radiation level of the building interior walls low, future disposal costs should be minimal.

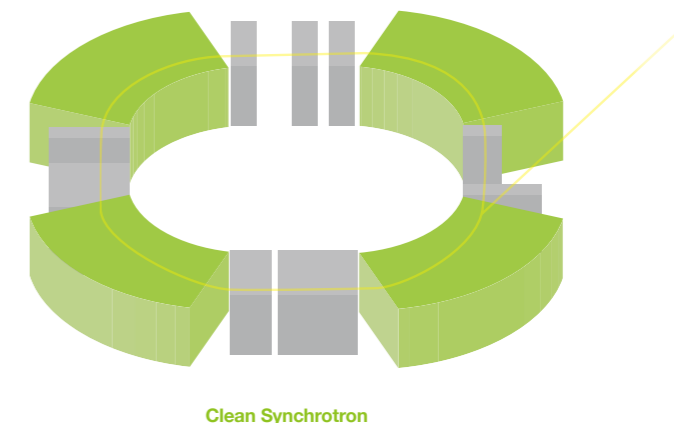

# Reliable Solution

From system installation to daily operation and system maintenance, Hitachi, with over 25 years of experience with particle therapy, has continually refined support services to ensure owners of highly reliable system operation. Continuous improvement has led to on-time installation, smooth/stable ramp-up, and excellent system uptime.

## High System Availability\*

With proven system availability, Hitachi is proud of its strong and successful system operation experience. Our high standards of engineering and manufacturing quality, coupled with seamless teamwork between the on-site maintenance team and our 24/7 remote maintenance service, yield industry-leading reliability so customers can focus on treating patients.

\*Uptime guarantee of the system is optional.



24/7 remote maintenance by Hitachi service team

## Dedicated Customer Training


Utilizing a vast user network comprised of the world's top particle therapy leaders, new Hitachi customers receive expert staff training before treatments begin, in addition to on-site training. Hitachi provides the support needed to open each facility with confidence and to maximize operational success.



On-site customer training  
Courtesy of St. Jude Children's Research Hospital

## Upgrade

Upgrades to expand the functionality of the system are developed and proposed throughout the lifetime of the system. As innovations are introduced, Hitachi customers are ensured that their technology investment will always be state-of-the-art.

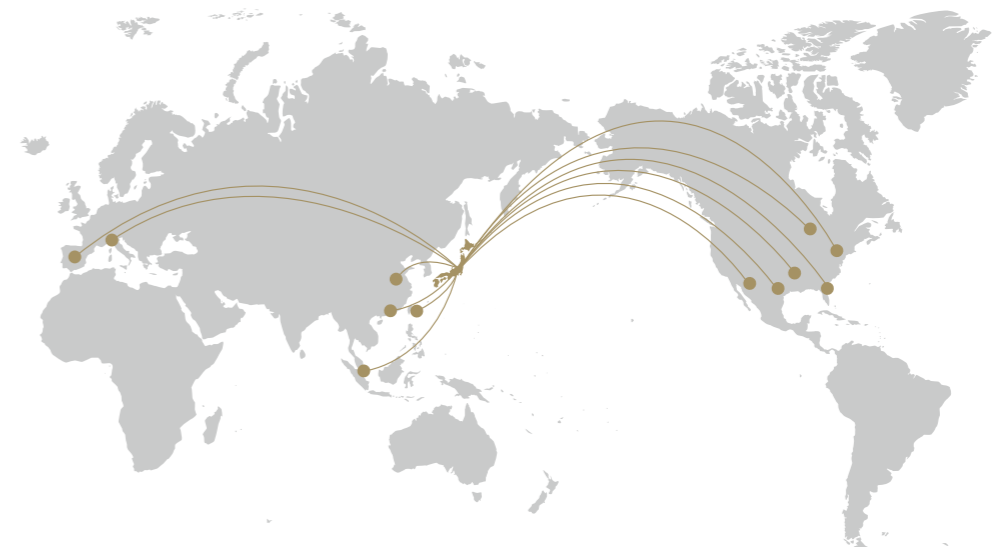


# Versatility in Proton Therapy

Hitachi is focusing on the development of technology to enhance versatility in proton therapy - aiming to expand treatment applications according to the needs of customers. Current Hitachi customers provide patients with a wide range of treatments utilizing today's advanced applications – Intensity Modulated Proton Therapy (IMPT), Real-Time Gated Proton Therapy (RGPT) and Cone Beam CT imaging for precise patient positioning and adaptive therapy.

## Proton Therapy system

■ We offer a wide range of treatment room configurations to meet our customers' needs.

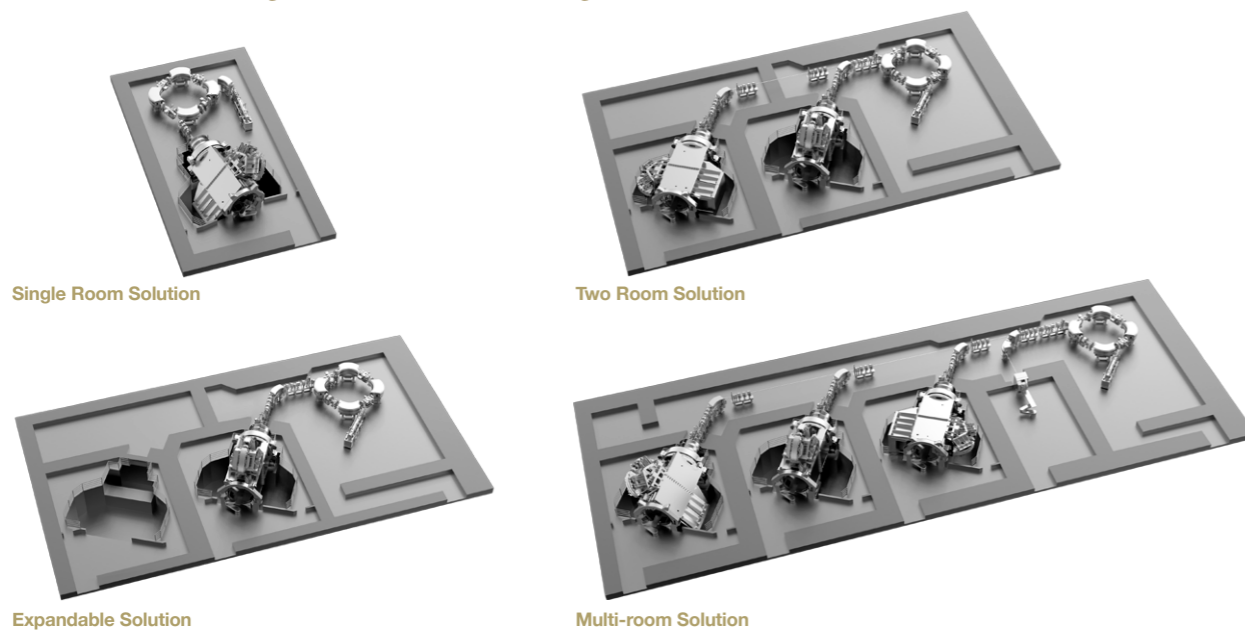

## Treatment Room Type Lineup

■ **360-Degree Rotating Gantry Treatment Room**  
This treatment room enables beam delivery from all directions over 360 degrees without changing the patient's position. With its spacious gantry interior, non-coplanar irradiation from the cranial direction is supported, and CBCT is provided as a standard function.

■ **Fixed Beam Treatment Room**  
This treatment room allows beam delivery from a single direction using a fixed-position nozzle.




360-Degree Rotating Gantry Treatment Room  
(Courtesy of National Cancer Centre Singapore)

※Specifications are subject to change without notice.

# World-Leading Carbon-Ion Therapy

## Most SELECTED Carbon-Ion Therapy Technology

Hitachi's Carbon-Ion Therapy system has large market share in numbers of facilities (in operation and under construction), and a proven track record, having treated 65% of all patients that have received Carbon-Ion Therapy in the world.

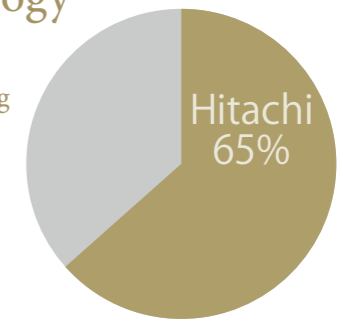

Patients treated with Carbon-Ion Therapy worldwide  
Source: PTCOG Patient Statistics

## Carbon-Ion Therapy System

Due to its larger mass, Carbon-Ions have 3x higher relative biological effectiveness (RBE) than photons or protons. Higher RBE means that patients need fewer fractions to administer the same amount of radiation dose with Carbon-Ion Therapy, and Carbon has demonstrated greater effectiveness on radio-resistant tumors. Hitachi has been supporting nine Carbon-Ion therapy facilities worldwide. In October 2018, Osaka Heavy Ion Therapy Center, Japan, started operation on schedule. In addition, the heavy ion therapy system delivered to Taipei Veterans General Hospital in May 2023 commenced the first Carbon-Ion therapy treatment in Taiwan. This also marked Hitachi's first operation of a Carbon-Ion therapy system overseas.



Osaka Heavy Ion Therapy Center

## Hybrid Therapy System

Utilizing decades of experience, Hitachi has integrated technologies to offer a hybrid therapy system, capable of irradiating with proton and carbon. In May 2019, Hitachi was awarded a contract to deliver the first hybrid system outside Japan by Xuzhou Proton and Heavy Ion Hospital, China. Furthermore, Hitachi can also offer systems to generate beams other ions for future clinical applications or for research purposes, according to customer needs. Following this, projects are currently underway for a hybrid system for Heyou International Hospital in China, as well as for the first hybrid system project in the United States in North America.



Hybrid Therapy System (model)

# Join Hitachi's Global Particle Therapy Network

Facility name (treatment start year) Facility information is current as of March 2026.

Hitachi holds regular users meetings with global particle therapy leaders who have selected our proton, carbon, and fine-tune the Hitachi R&D roadmap to enhance existing systems and integrate new capabilities into the opportunity to Hitachi's innovations in particle therapy.

and hybrid solutions. Here, we discuss the most advanced clinical and operational experiences and objectives future product and technology pipeline. It is a highly valued exchange of ideas, providing a unique forum and

Discover the right particle therapy solution for your patients and join Hitachi's global particle therapy network

of leading radiation oncology centers.



MD Anderson Cancer Center Proton Therapy Center Houston, TX, USA (2006)




Mayo Clinic Rochester Campus Rochester, MN, USA (2015)



St. Jude Children's Research Hospital Memphis, TN, USA (2015)



Mayo Clinic Phoenix Campus Phoenix, AZ, USA (2016)

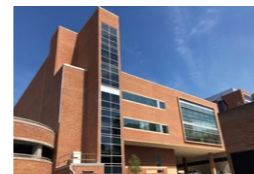

Sibley Memorial Hospital, a member of Johns Hopkins Medicine Washington DC, USA (2019)



National Institutes for Quantum and Radiological Science and Technology National Institute of Radiological Sciences Chiba, Japan (1994)



University of Tsukuba Hospital Proton Beam Therapy Center Ibaraki, Japan (2001)



Hyogo Ion Beam Medical Center Hyogo, Japan (2001)



Shizuoka Cancer Center Shizuoka, Japan (2003)



Southern TOHOKU Proton Therapy Center Fukushima, Japan (2008)



MD Anderson Cancer Center Proton Therapy Center \*Additional construction Houston, TX, USA (2024)



Mayo Clinic Jacksonville Campus Jacksonville, FL, USA (Under Construction)



University of Wisconsin Health Madison, WIS, USA (Under Construction)



Mayo Clinic Rochester Campus \*Additional construction Rochester, MN, USA (Under construction)



Gunma University Heavy Ion Medical Center Gunma, Japan (2010)



Fukui Prefectural Hospital Proton Therapy Center Fukui, Japan (2011)



Medipolis Medical Research Institute Medipolis Proton Therapy and Research Center Kagoshima, Japan (2011)



Nagoya Proton Therapy Center Aichi, Japan (2013)



SAGA Heavy Ion Medical Accelerator in Tosu Kyushu International Heavy Ion Beam Therapy Center Saga, Japan (2013)



HKSH Eastern Hospital HKSH Proton Therapy Centre Hong Kong (2023)



Taipei Veterans General Hospital Heavy Ion Therapy Center Taiwan (2023)



National Cancer Centre Singapore (2023)



Xuzhou Proton and Heavy Ion Hospital, Xuzhou City, China (Under Construction)



Heyou International Hospital Project Proton & Heavy Ion Center Foshan, China (Under Construction)



Hokkaido University Hospital Proton Beam Therapy Center Hokkaido, Japan (2014)



Tsuyama Chuo Hospital Okayama University Proton Beam Okayama, Japan (2016)



Hakuhoikai Osaka Proton Therapy Clinic Osaka, Japan (2017)



Hyogo Ion Beam Medical Center Kobe Proton Center Hyogo, Japan (2017)



Osaka Heavy Ion Therapy Center Osaka, Japan (2018)



Clínica Universidad de Navarra Madrid, Spain (2020)



Centro Nazionale di Adroterapia Oncologica (CNAO) Pavia, Italy (Under Construction)



Kyoto Prefectural University of Medicine Nagamori Memorial Center of Innovative Cancer Therapy and Research Kyoto, Japan (2019)



Shonan Kamakura General Hospital Kanagawa, Japan (2022)



University of Tsukuba Hospital Proton Beam Therapy Center \*Additional construction Ibaraki, Japan (2025)



Tokyo Metropolitan Komagome Hospital Tokyo, Japan (Under Construction)

